


SPACE SAVER TOILET SEAT RAISER

RG515

September 2019

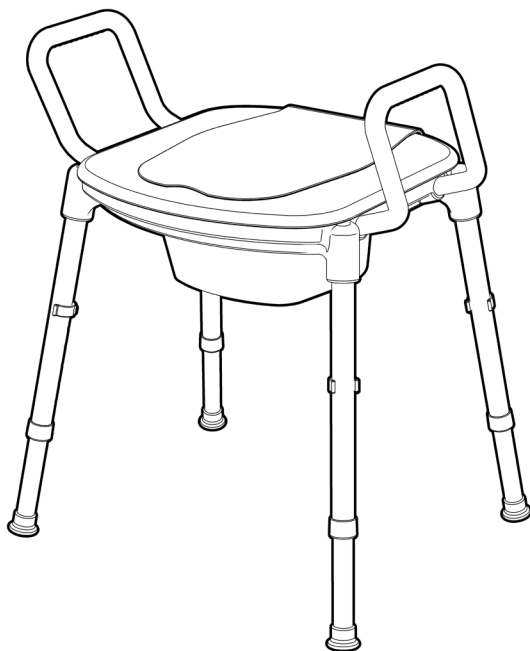

PATENT PENDING

USER MANUAL

Please read and keep these instructions



REDGUM Brand owns the copyright to the Space Saver Toilet Seat Raiser user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand** 2019




REDGUM Brand is part of **FOR-DE Group Pty Ltd**

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **ASSEMBLY** - UNPACK and INSPECT SUPPLIED PARTS
4. **ASSEMBLY** - LEG ASSEMBLY
5. **ASSEMBLY** - LEG INSTALLATION
6. **ASSEMBLY** - HANDLE INSTALLATION
7. **USAGE GUIDELINES** - SPLASHGUARD
8. **USAGE GUIDELINES** - DO's + DON'Ts
9. **PRODUCT CARE** - INSPECTIONS / CLEANING
10. **WARRANTY**
11. **REDGUM BRAND** PRODUCT RANGE
12. **NOTES**

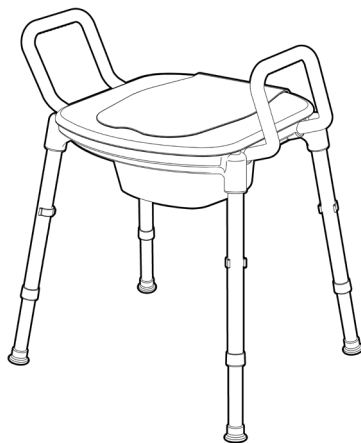

INTRODUCTION

Thank you for purchasing this quality **REDGUM Brand** product.

If you have any issues or questions about the **Space Saver Toilet Seat Raiser** please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180

F: +61 8 9248 9452

E: orders@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.


Only use your **Space Saver Toilet Seat Raiser** for the purpose for which it is intended, which is as a seating solution that can adjust to suit the users individual requirements.

IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

Maximum User Weight: 130kg


Exceeding the weight limit could cause damage to the product and could put the user at risk.

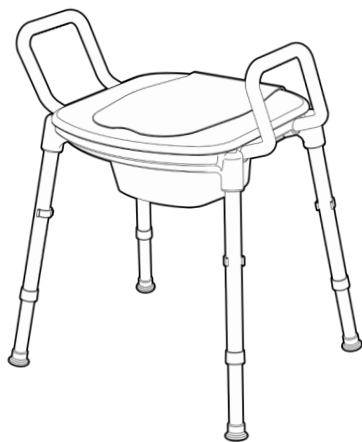

PRODUCT OVERVIEW

FEATURES

Modern / Functional / Compact design
Ideal for use in confined spaces
Easy to assemble - No tools required
Adjustable seat height and angle with sure lock push pins
Contoured toilet seat
Toilet seat flap
Removable splashguard
Raised handles with ribbed finger grips
Flexible PVC leg tips
Strong lightweight construction
Corrosion free materials
Life time guarantee on corrosion
Designed for easy cleaning

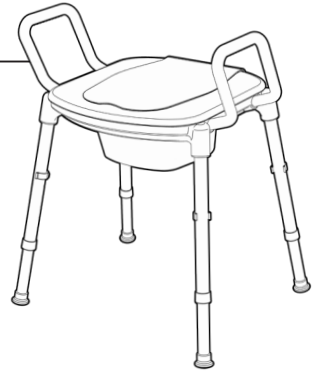

SPECIFICATIONS

(All dimensions shown are nominal)

Overall Width: **540mm** (at lowest setting)
Overall Depth: **440mm** (at lowest setting)
Overall Height: **655mm** (at lowest setting)
Seat Width: **470mm** (between armrests)
Seat Depth: **400mm**
Seat Height: **500 - 625mm: 6 settings** (floor to top of seat)
Hole 1: **500mm**
Hole 2: **525mm**
Hole 3: **550mm**
Hole 4: **575mm**
Hole 5: **600mm**
Hole 6: **625mm**
Handle Height: **150mm** (seat to top of armrest)

Product Weight: 3.5kg


Maximum User Weight: 130kg

Please contact **REDGUM Brand** if you would like any further information on your **Space Saver Toilet Seat Raiser**.

ASSEMBLY - UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:

PARTS:

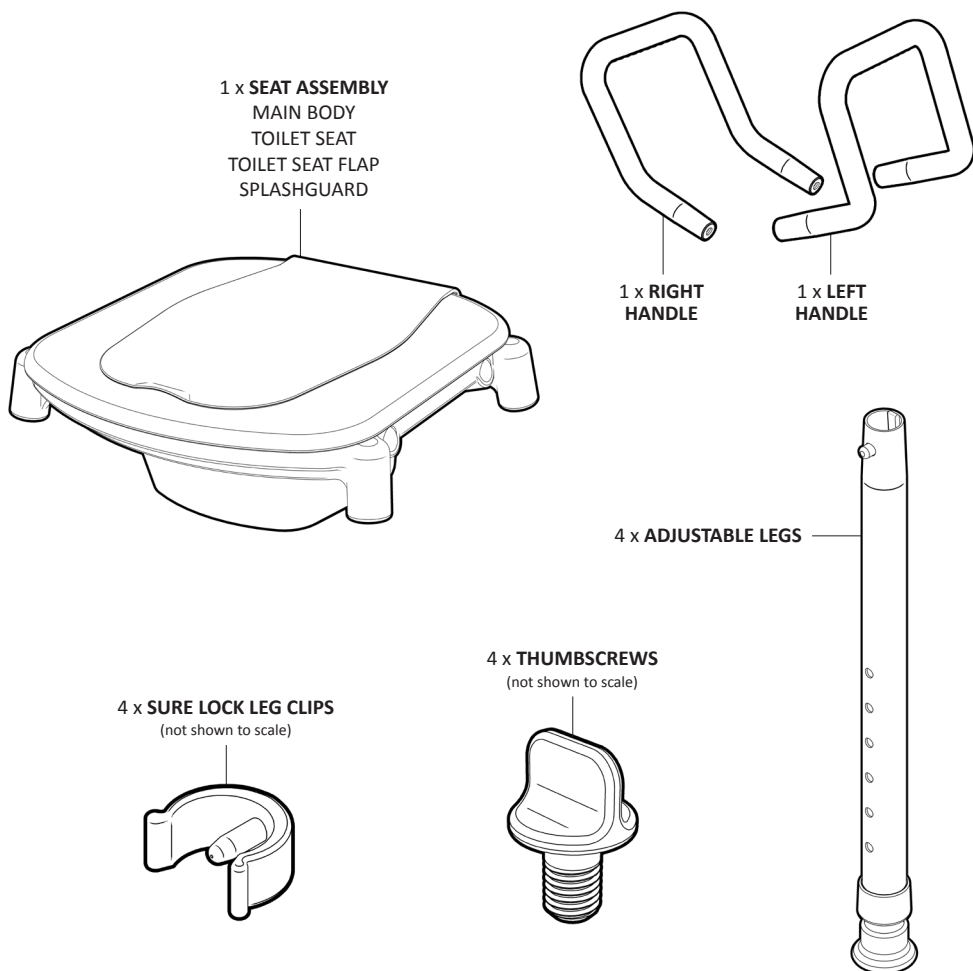

IMPORTANT NOTE


Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE YOUR PRODUCT** and contact your supplier for support.

ASSEMBLY - LEG ASSEMBLY

Each adjustable leg needs to be assembled prior to fitting to main body:

1. Pull out the inner leg.
2. Line up the holes at the required setting (setting can be changed at any time).
3. Insert sure lock push pin into hole and push clip until it locates around tube.
4. **Repeat this process on all 4 adjustable legs.**

Adjustable legs are now ready to be fitted to main body.

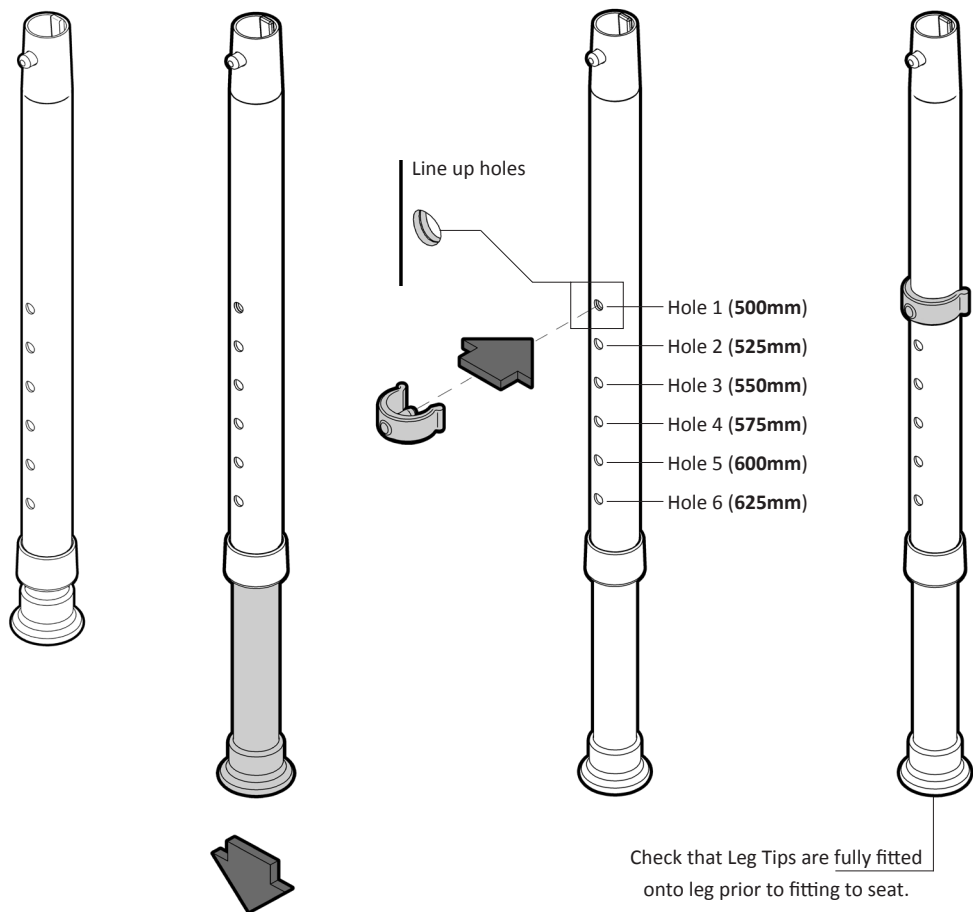


To adjust the legs after they have been fitted to main body:

1. Remove sure lock push pin.
2. Adjust leg to required setting (line up holes).
3. Insert sure lock push pin into hole and push clip until it locates around tube.


ASSEMBLY - LEG INSTALLATION

Each adjustable leg needs to be installed to main body:




1. Place seat upside down as shown below.
2. Insert assembled leg into socket.
3. Line up red press button with hole in wall of socket.
4. Depress press button and push leg down into socket.
5. Ensure head of button protrudes out of hole.
6. **Repeat process for all 4 adjustable legs.**

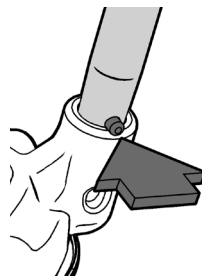

Line up press button
with hole

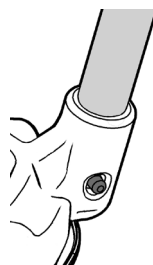

IMPORTANT NOTE:
Press button head **MUST**
protrude out of hole

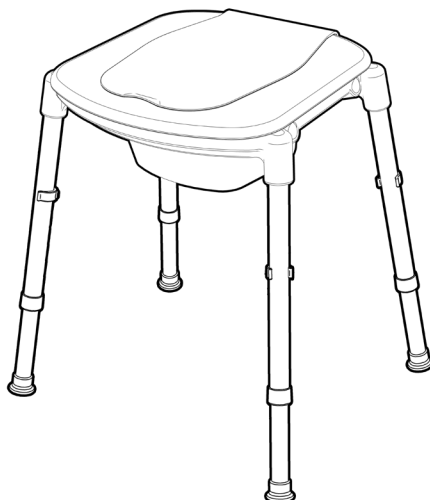
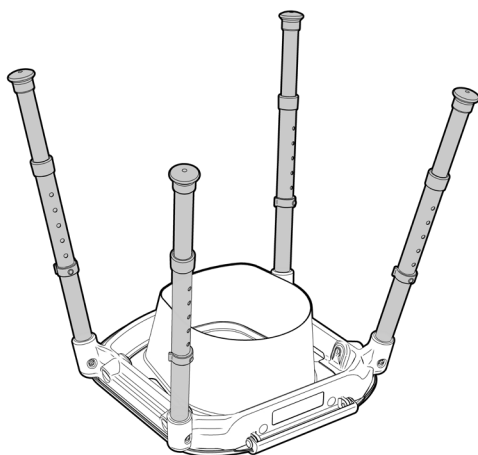
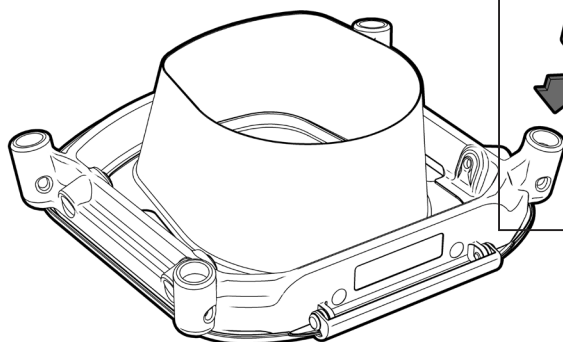


Flip Space Saver Toilet Seat Raiser over onto feet.

ASSEMBLY - HANDLE INSTALLATION

Each handle needs to be installed to main body:

1. Ensure angled front of handle is **facing toward front of main body**.
2. Insert ends of handle into sockets on side of main body.
3. Insert thumbscrew into hole on underside of main body.
4. Hand tighten thumbscrew until slight resistance is felt.
(DO NOT OVERTIGHTEN)

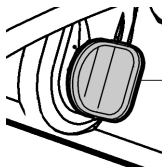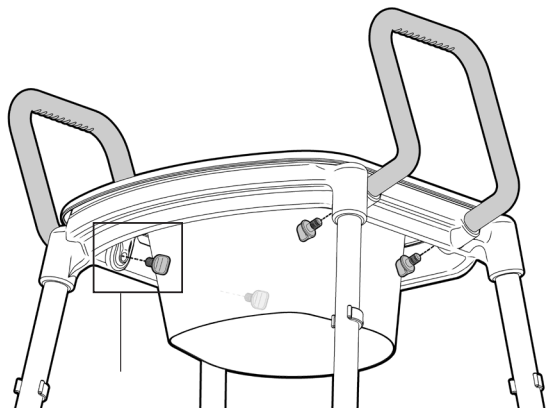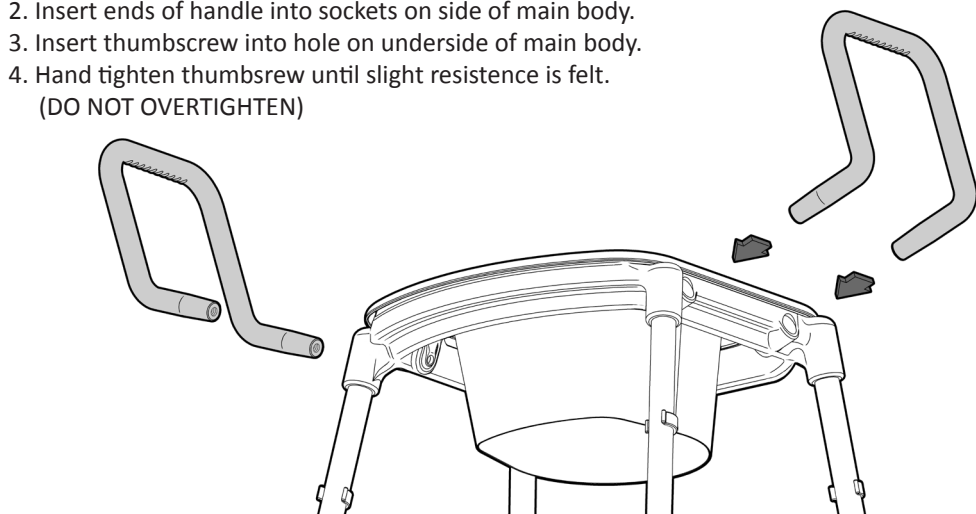

IMPORTANT NOTE:
DO NOT overtighten
thumbscrew





Product is now ready to use.
(Adjust seat height as required.)


USAGE GUIDELINES - SPLASHGUARD

The Space Saver Toilet Seat Raiser has a removable splashguard.
To remove the splashguard:

1. Flip up the toilet seat flap and toilet seat.
2. Using the cut outs on either side, pull out the splashguard to clean or remove if not required.

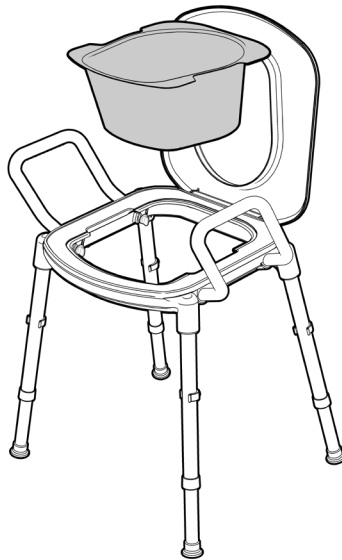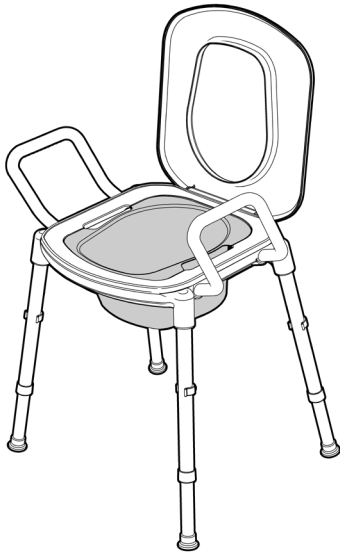

When re-installing the splashguard, ensure that the tapered front of the splashguard is facing toward the front of the Space Saver Toilet Seat Raiser.

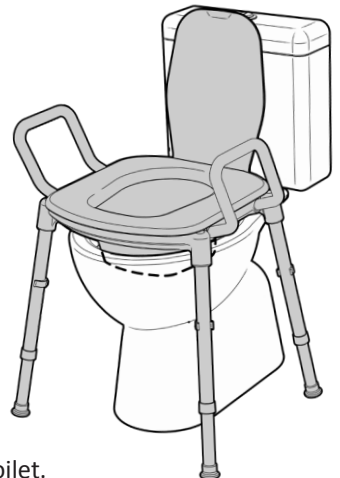

Centrally position the Space Saver Toilet Seat Raiser over toilet.


USAGE GUIDELINES - DO's + DON'TS

Please follow these guidelines to get the optimal benefit from your Space Saver Toilet Seat Raiser.

Failure to follow these guidelines may result in damage to the Space Saver Toilet Seat Raiser and put the user at risk.

130kg Maximum User Weight.

Ensure ALL FOUR LEGS remain on ground at all times when Space Saver Toilet Seat Raiser is occupied.

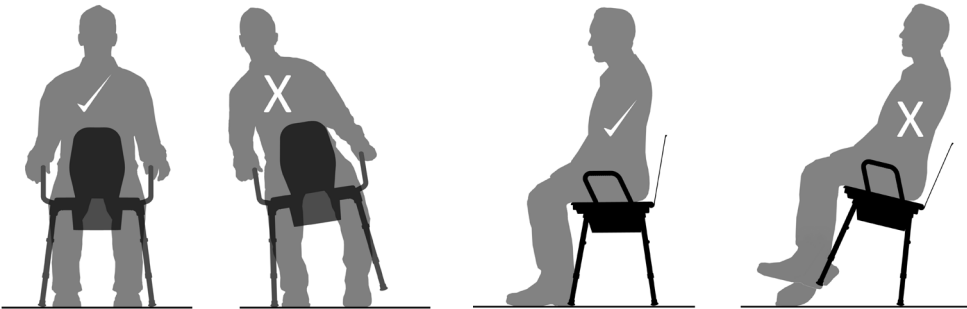


Do not lean too far to the side or backward causing the legs to leave the ground.

SITTING GUIDELINES

The Space Saver Toilet Seat Raiser can be set up in various ways to suit the individuals requirements.

Adjust the seat height to line up with back of the users knees.
(Refer to Page 4 for information on leg adjustment)

To assist the user with getting on and off of the Space Saver Toilet Seat Raiser, extend the rear legs to open up the seat angle.
(Maximum 90°)

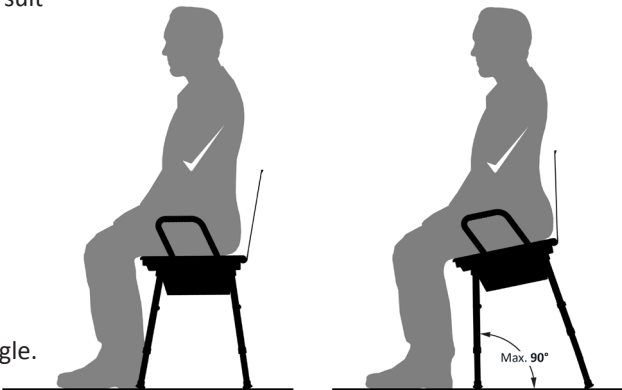

PRODUCT CARE - INSPECTIONS / SERVICING / CLEANING

INSPECTIONS

We suggest that you check your Space Saver Toilet Seat Raiser visually periodically for signs of damage or wear prior to use. Overall check that no components are loose or worn.



Withdraw the Space Saver Toilet Seat Raiser immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT NOTE

Do not fit any parts or accessories to your Space Saver Toilet Seat Raiser that are manufactured by another company. To do so will void your warranty and may seriously compromise your safety.

CLEANING

Clean your Space Saver Toilet Seat Raiser using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the Space Saver Toilet Seat Raiser and should not be used.



WARRANTY

Your REDGUM Brand **RG515 - Space Saver Toilet Seat Raiser** is covered by a **12 Month Warranty** from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

If you have any issues or questions about your **Space Saver Toilet Seat Raiser** please contact your supplier or REDGUM Brand direct on:


T: +61 8 9248 4180

F: +61 8 9248 9452

E: orders@for-de.com.au

A: 1 Business Way, Malaga
Western Australia 6090


**Please keep this User Manual
in a safe place for future reference.**

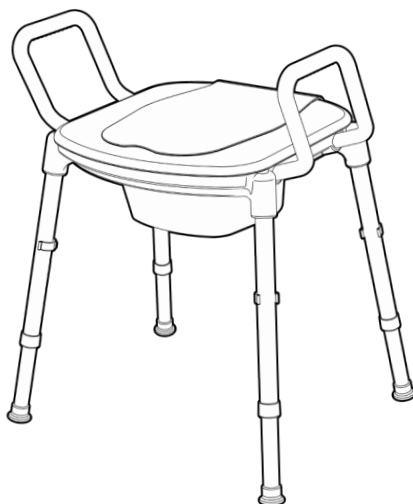

REDGUM Brand - Product Range

REDGUM Brand offer a comprehensive range of products including:

Seat Walkers
Side Folding Seat Walkers
Walking Frames
Wheelchairs
Walking Sticks + Quads
Crutches
Reachers
Shower / Bathroom products
Toilet products
Cushions: Air / Foam
Electric Lift Chairs
Electric Beds
Portable / Fixed Aluminium Ramps
Exercisers
Personal Alarms

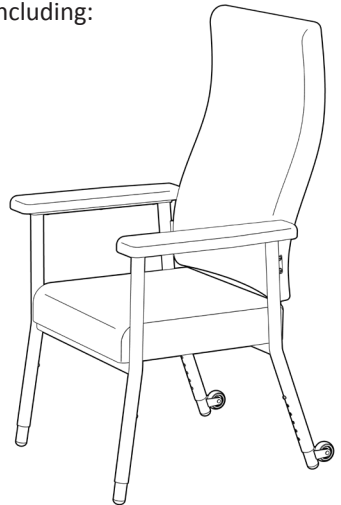

REDGUM Brand stock a large range of Spare Parts for all REDGUM brand products.

Please contact REDGUM Brand for further information:


T: +61 8 9248 4180

F: +61 8 9248 9452

E: orders@for-de.com.au

A: 1 Business Way, Malaga Western Australia 6090

W: www.redgumbrand.com.au

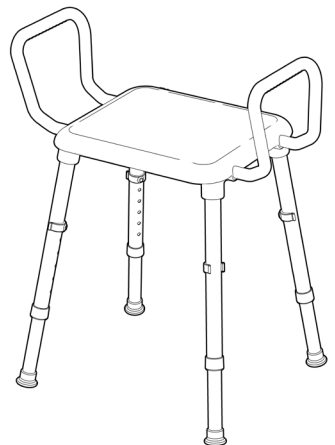

NOTES



DISTRIBUTOR:

REDGUM BRAND
*Daily Living and
Lifestyle Products*



Proudly Australian Owned and Operated