


# SPACE SAVER SHOWER CHAIR

## RG555H

September 2019

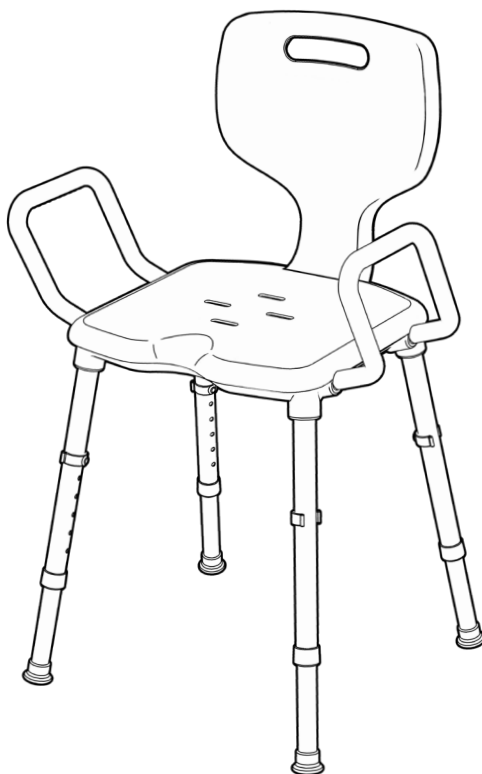

**PATENT PENDING**

## USER MANUAL

---

Please read and keep these instructions



**REDGUM Brand** owns the copyright to the **Space Saver Shower Chair** user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

**REDGUM Brand** strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand** 2019



**REDGUM Brand** is part of **FOR-DE Group Pty Ltd**

[redgumbrand.com.au](http://redgumbrand.com.au)

# CONTENTS

---

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **ASSEMBLY** - UNPACK and INSPECT SUPPLIED PARTS
4. **ASSEMBLY** - LEG ASSEMBLY
5. **ASSEMBLY** - LEG INSTALLATION
6. **ASSEMBLY** - HANDLE INSTALLATION
7. **ASSEMBLY** - BACKREST INSTALLATION
8. **USAGE GUIDELINES** - DO's + DON'Ts
9. **PRODUCT CARE** - INSPECTIONS / CLEANING
10. **WARRANTY**
11. **REDGUM BRAND** PRODUCT RANGE
12. **NOTES**



# INTRODUCTION

---

Thank you for purchasing this quality **REDGUM Brand** product.

If you have any issues or questions about the **Space Saver Shower Chair** please contact your supplier or REDGUM Brand direct:

T: +61 8 9248 4180

F: +61 8 9248 9452

E: [orders@for-de.com.au](mailto:orders@for-de.com.au)

A: 1 Business Way, Malaga Western Australia 6090

## INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your **Space Saver Shower Chair** for the purpose for which it is intended, which is as a seating solution that can adjust to suit the users individual requirements.

### IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

**Maximum User Weight: 130kg**

Exceeding the weight limit could cause damage to the product and could put the user at risk.



# PRODUCT OVERVIEW

---

## FEATURES

Modern / Functional / Compact design  
Ideal for use in confined spaces  
Easy to assemble - No tools required  
Adjustable seat height and angle with sure lock push pins  
Contoured seat with drainage holes  
Contoured backrest with carry handle  
Raised handles with ribbed finger grips  
Flexible PVC leg tips  
Strong lightweight construction  
Corrosion free materials  
Life time guarantee on corrosion  
Designed for easy cleaning



## SPECIFICATIONS

(All dimensions shown are nominal)

Overall Width: **515mm** (at lowest setting)  
Overall Depth: **450mm** (at lowest setting)  
Overall Height: **825mm** (at lowest setting)  
Seat Width: **470mm** (between armrests)  
Seat Depth: **355mm**  
Seat Height: **485 - 610mm: 6 settings** (floor to top of seat)  
Hole 1: **485mm**  
Hole 2: **510mm**  
Hole 3: **535mm**  
Hole 4: **560mm**  
Hole 5: **585mm**  
Hole 6: **610mm**  
Handle Height: **155mm** (seat to top of armrest)

Product Weight: 3.1kg


Maximum User Weight: 130kg

Please contact **REDGUM Brand** if you would like any further information on your **Space Saver Shower Chair**.

# ASSEMBLY - UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:

## PARTS:

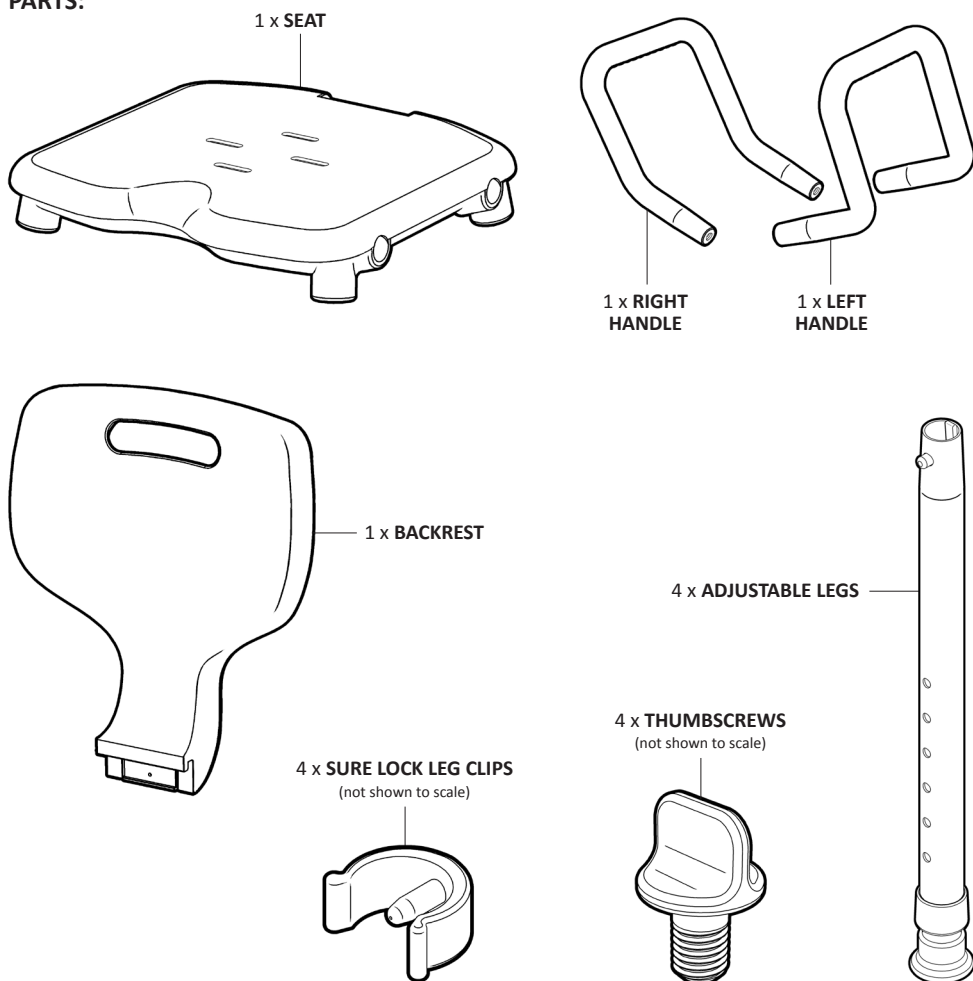

## IMPORTANT NOTE


Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE YOUR PRODUCT** and contact your supplier for support.

# ASSEMBLY - LEG ASSEMBLY

Each adjustable leg needs to be assembled prior to fitting to seat:

1. Pull out the inner leg.
2. Line up the holes at the required setting (setting can be changed at any time).
3. Insert sure lock leg clip into hole and push clip until it locates around tube.
4. **Repeat this process on all 4 adjustable legs.**

Adjustable legs are now ready to be fitted to seat.

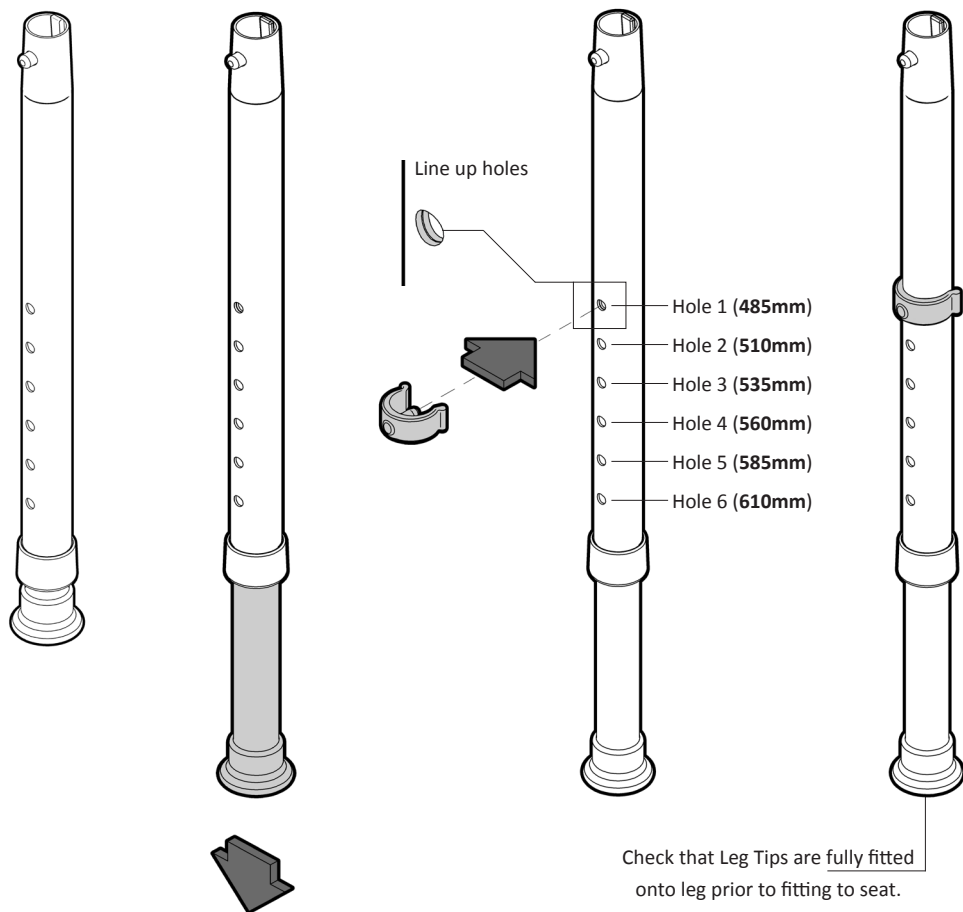


To adjust the legs after they have been fitted to seat:

1. Remove sure lock leg clip.
2. Adjust leg to required setting (line up holes).
3. Insert sure lock leg clip into hole and push clip until it locates around tube.


# ASSEMBLY - LEG INSTALLATION

Each adjustable leg needs to be installed to seat:



1. Place seat upside down as shown below.
2. Insert assembled leg into socket.
3. Line up red press button with hole in wall of socket.
4. Depress press button and push leg down into socket.
5. Ensure head of button protrudes out of hole.
6. Repeat process for all 4 adjustable legs.

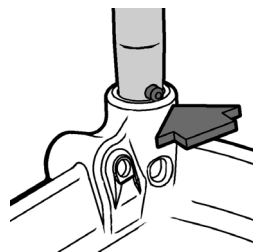

Line up press button with hole

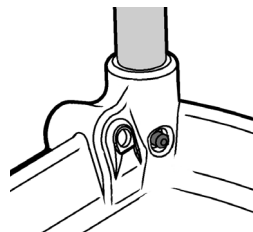

**IMPORTANT NOTE:**  
Press button head **MUST** protrude out of hole


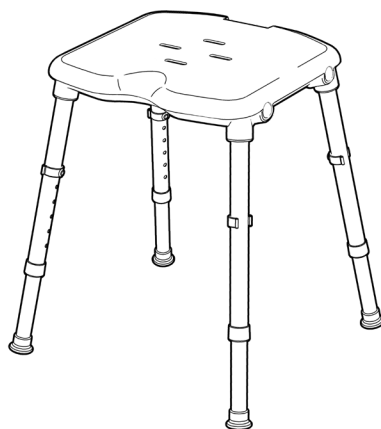
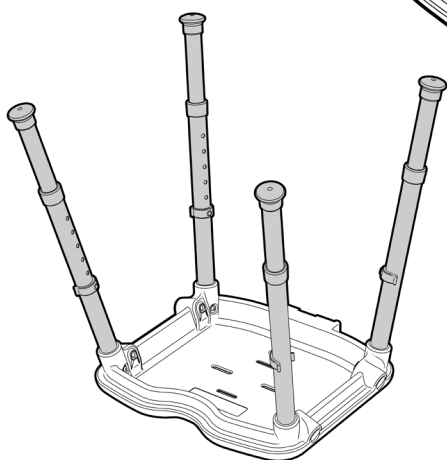


Flip Space Saver Shower Chair over onto feet.  
(Adjust seat height as required.)


## ASSEMBLY - HANDLE INSTALLATION

Each handle needs to be installed to seat (if required):

1. Ensure angled front of handle is **facing toward front of seat**.
2. Insert ends of handle into sockets on side of seat.
3. Insert thumbscrew into hole on underside of seat.
4. Hand tighten thumbscrew until slight resistance is felt.  
(DO NOT OVERTIGHTEN)

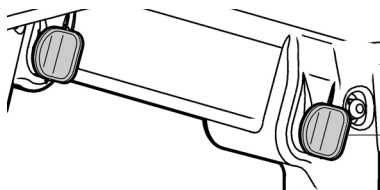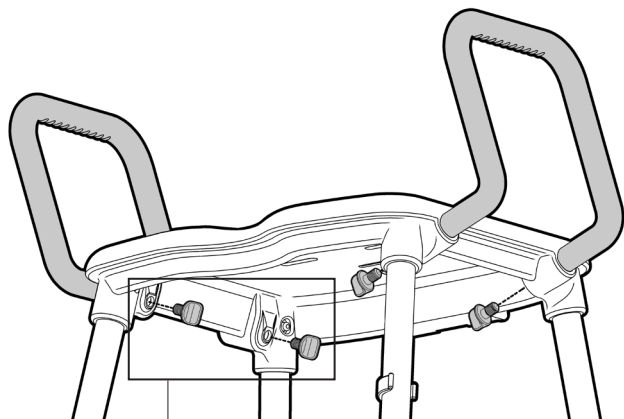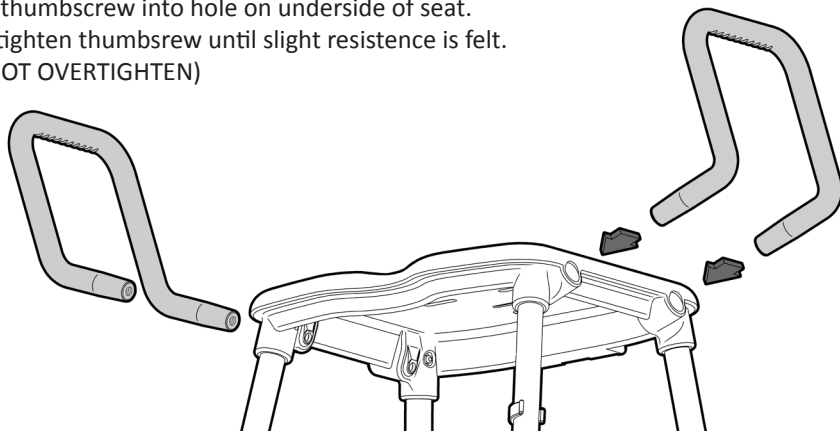


**IMPORTANT NOTE:**  
DO NOT overtighten  
Thumbscrews

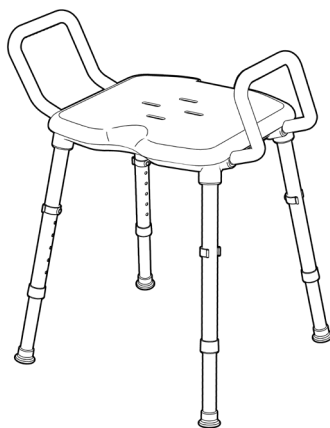

# ASSEMBLY - BACKREST INSTALLATION

The backrest needs to be installed to seat:

1. Locate the backrest onto seat as shown.
2. Press down on backrest until a 'CLICK' is heard.
3. Visually check to ensure the backrest has fully located into position.

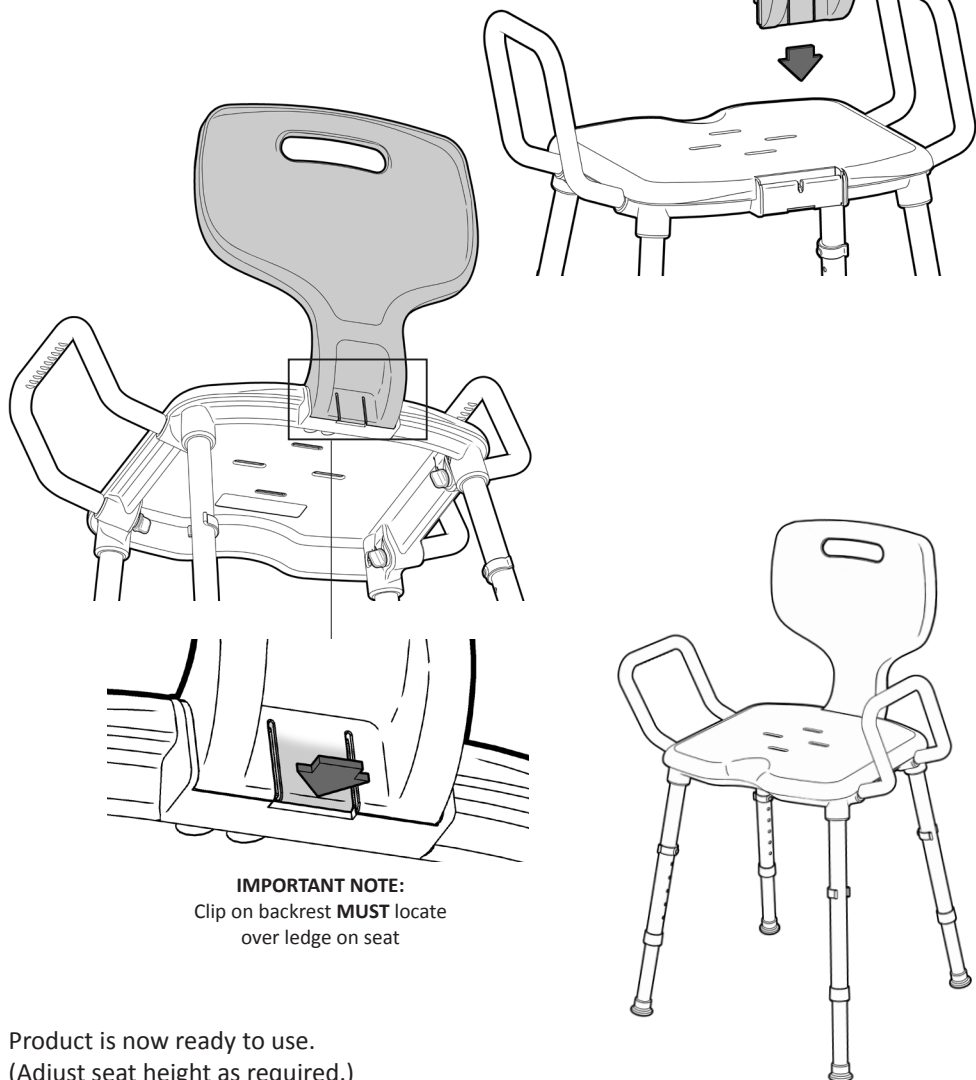

Product is now ready to use.  
(Adjust seat height as required.)


# USAGE GUIDELINES - DO's + DON'TS

Please follow these guidelines to get the optimal benefit from your Space Saver Shower Chair.

Failure to follow these guidelines may result in damage to the Space Saver Shower Chair and put the user at risk.

**130kg** Maximum User Weight.

Ensure ALL FOUR LEGS remain on ground at all times when Space Saver Shower Chair is occupied.

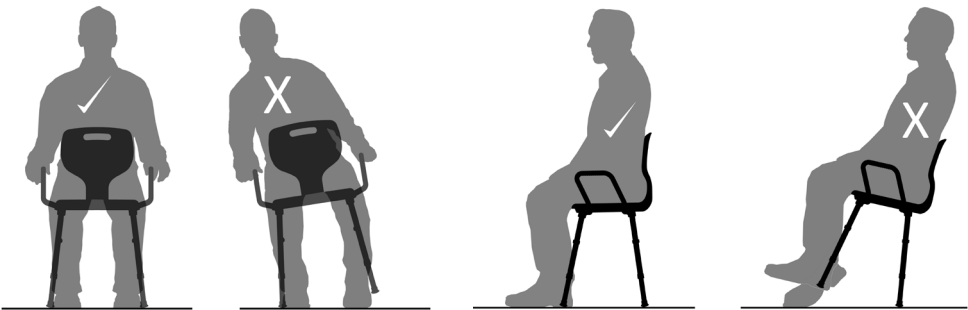


Do not lean too far to the side or backward causing the legs to leave the ground.

## SITTING GUIDELINES

The Space Saver Shower Chair can be set up in various ways to suit the individuals requirements.

Adjust the seat height to line up with back of the users knees.  
(Refer to Page 4 for information on leg adjustment)

To assist the user with getting on and off of the Space Saver Shower Chair, extend the rear legs to open up the seat angle.  
(Maximum 90°)

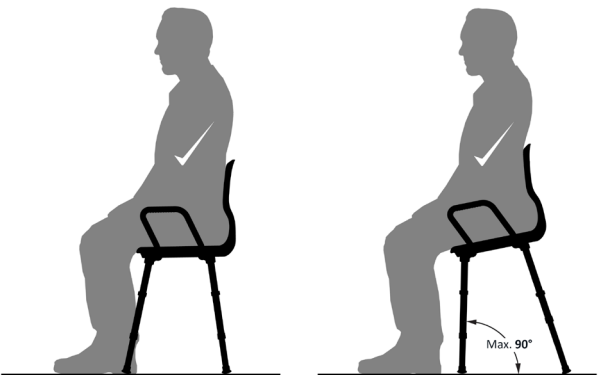

# PRODUCT CARE - INSPECTIONS / SERVICING / CLEANING

---

## INSPECTIONS

We suggest that you check your Space Saver Shower Chair visually periodically for signs of damage or wear prior to use. Overall check that no components are loose or worn.



**Withdraw the Space Saver Shower Chair immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## IMPORTANT NOTE

Do not fit any parts or accessories to your Space Saver Shower Chair that are manufactured by another company. To do so will void your warranty and may seriously compromise your safety.

## CLEANING

Clean your Space Saver Shower Chair using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the Space Saver Shower Chair and should not be used.



# WARRANTY

---

Your REDGUM Brand **RG555H - Space Saver Shower Chair** is covered by a **12 Month Warranty** from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

If you have any issues or questions about your **Space Saver Shower Chair** please contact your supplier or REDGUM Brand direct on:

T: +61 8 9248 4180

F: +61 8 9248 9452

E: [orders@for-de.com.au](mailto:orders@for-de.com.au)

A: 1 Business Way, Malaga  
Western Australia 6090

**Please keep this User Manual  
in a safe place for future reference.**



# REDGUM Brand - Product Range

---

REDGUM Brand offer a comprehensive range of products including:

**Seat Walkers**

**Side Folding Seat Walkers**

**Walking Frames**

**Wheelchairs**

**Walking Sticks + Quads**

**Crutches**

**Reachers**

**Shower / Bathroom products**

**Toilet products**

**Cushions: Air / Foam**


**Electric Lift Chairs**

**Electric Beds**

**Portable / Fixed Aluminium Ramps**

**Exercisers**

**Personal Alarms**

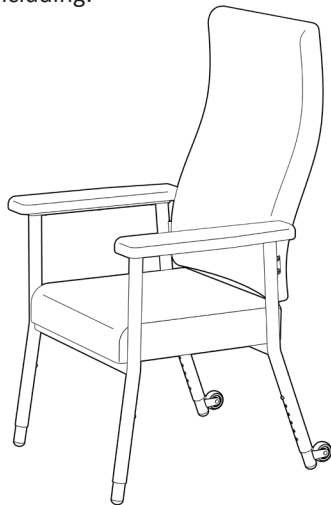

REDGUM Brand stock a large range of Spare Parts for all REDGUM brand products.

Please contact REDGUM Brand for further information:


T: +61 8 9248 4180

F: +61 8 9248 9452

E: [orders@for-de.com.au](mailto:orders@for-de.com.au)

A: 1 Business Way, Malaga Western Australia 6090

W: [www.redgumbrand.com.au](http://www.redgumbrand.com.au)

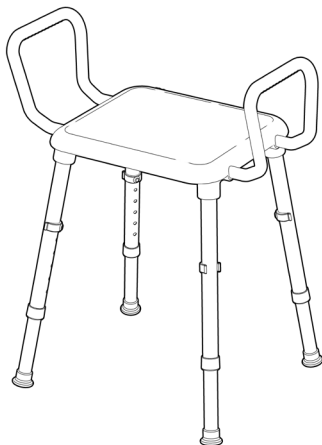

# NOTES

---



**DISTRIBUTOR:**

**REDGUM** *BRAND*  
*Daily Living and  
Lifestyle Products*



**Proudly Australian Owned and Operated**

**redgumbrand.com.au**